



# E4D Technologies

Doctors with an E4D system can send digital impressions directly to Williams Dental Laboratory through E4D Sky.

## To send a file:

- 1) Setup, scan, and mark your margin. Click 'E4D SKY' on the top right hand corner.
- 2) Click 'Export' and Send Buccal Bite Case.
- 3) Before you click 'Submit' make sure the boxes are selected to save and email the STL files.
- 4) Once you click 'Submit' a new screen asking to upload the STL files will appear.  
Select your prep, opposing, or buccal bites.
- 5) When you click 'Browse', a folder containing the STL files will appear, then select the coordinating files.
- 6) Once all files have been uploaded, enter your email address and Williams Dental Laboratory email: [digitalscans@williamsdentallab.com](mailto:digitalscans@williamsdentallab.com)
- 7) **IMPORTANT NOTE:**  
Do not close the E4D software or turn off the system until the case has been fully transferred.